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User Manual - Bipolar Forceps for Electrosurgery

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WEM accessories for electrosurgery guarantee your equipment's optimal performance, precision and efficiency.

The items have been especially designed for use with electrosurgical unit manufactured by either WEM or competitors. There are nearly 300 products, which are recommended for applications in a number different specialized medical fields.

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Forceps

Trade name

Bipolar forceps for electrosurgery

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WEM bipolar electrosurgical forceps are recommended for coagulation in minor, moderate and major surgical procedures.

Coupled with WEM electrosurgical generators or equipment from other suppliers, our bipolar forceps for electrosurgery aim to conduct the electrical current generated by electrosurgical units to the area to be clamped.

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Made with SGH 7C27 M02 stainless steel and insulated with polyamide (nylon), fixed with insulated material filled with XB313-2 Araldite resin.

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- Do not activate the generator until the electrosurgical forceps have touched the patient.
- Establish the power level that is required by the specific surgical procedure. Power levels are related to: coagulation time, tip dimension of forceps in use, amount of tissue to be clamped.
- The higher the power level, the lower the coagulation time. Forceps with thin tips require lower power levels. The larger the amount of tissue to be clamped, the higher the power required for coagulation.
- Before using the equipment, check the condition of the cable and accessories for insulation problems (dry, cracked, flawed), broken cable and connectors, and replace any defective item in order to avoid risk to patients and operators.
- Do not coil cables of accessories around metallic objects. This might induce potentially harmful currents in these objects and lead to shocks or burns to patients or the surgical team.
- Do not connect to the generator any accessory if it is wet or has humidity inside. There may be risk of electric shock.
- When more than one accessory is connected to the electrosurgical unit, care must be taken to keep the accessories that are not being used in a safe place. For instance, use a plastic or fabric bag that can be attached to surgical field and avoiding possible undesirable burns.
- Make sure the product is firmly connected to the correct power output of the electrosurgical unit.
- Never place the product on the patient's body when it is not being used during the surgical procedure.
- Do not use the product in the presence of flammable gases or substances.
- Keep the tips of electrosurgical forceps clean at all times by removing remains of burned tissue.

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- Connect the WEM Bipolar Forceps for Electrosurgery to the correct bipolar cable.
- Connect the bipolar forceps cable to the correct output shown on the electrosurgical unit panel.
- Check if the power level has been correctly set on the electrosurgical unit.
- Press the footswitch to operate the forceps.

Warning: Activate the electrosurgical unit only when the bipolar forceps have touched the tissue to be coagulated.

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Clean before sterilizing.
Sterilize prior to first use and after each subsequent use.

Cleaning methods

- Clean in running water and neutral soap
- Clean in germicide solution.
- Clean in degerming solution using ultrasound washer.

Sterilization methods

- Autoclave – at 134°C (274°F) for 15 minutes.
- Sterilization with ethylene oxide.
- Autoclave sterilization with formalin.
- Sterilization through immersion in germicide solution (enzymatic detergents)
- Sterilization with hydrogen peroxide (Sterrad)

WARNING: WEM does not recommend the use of Glutaraldehyde on stainless steel, tungsten wire accessories or those that have electrical terminals.

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Transport and store at room temperature in cool, dry areas sheltered from dust and humidity.

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This product has a warranty against manufacturing defects for 3 months from the issuance of the invoice.

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NON-STERILE MEDICAL
AND HOSPITAL PRODUCT.
STERILIZE
BEFORE USING.

Fabricante do material de uso médico
e detentor do registro /
Manufacturer of the product for
medical use and holder of
registration:

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Responsável Legal / Legal Representative:

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Warning: The use of any part, accessory or material not specified or set forth in this User Manual is the sole responsibility of the user.

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"WEM Equipamentos Eletrônicos Ltda. reserves the right to make any changes to the content provided in this manual as well as to the characteristics of the accessories, with no prior warning"